


# Attaching the Sports Lock

---




**1**

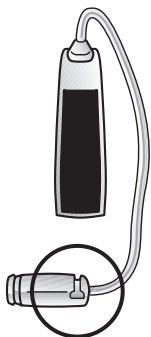


Locate the slit in the receiver end of the receiver tube.

**2**

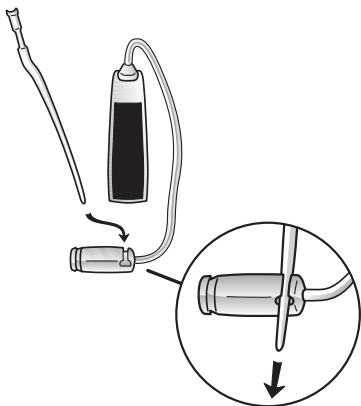


Slide the narrowest end of the sports lock into the slit starting from the side of the slit that is closest to the hearing instrument.

**3**

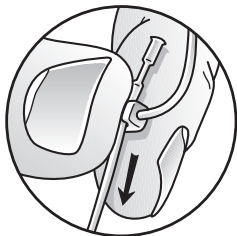

Place the receiver and sports lock between thumb and index finger and then pull the sports lock through.

**4**

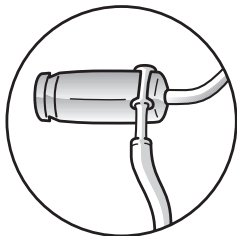


The sports lock is not fully inserted until the end of the sports lock is flush with the receiver.

# Removing the Sports Lock

---




**1**

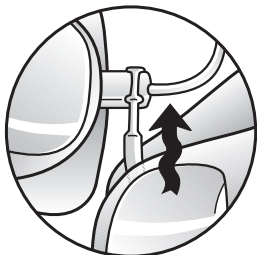


Hold the receiver in one hand and the sports lock in the other while pushing/wiggling the sports lock.

**2**

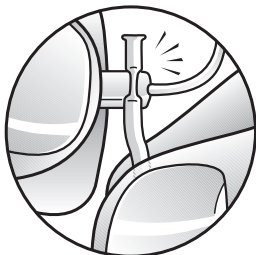

The wide part of the sports lock will snap out of place and the thin part of the sports lock will be positioned in the slit of the receiver end of the receiver tubing.

**3**

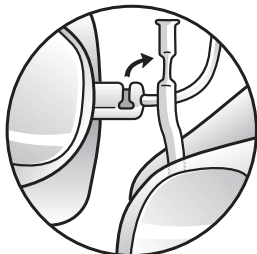

Lift the sports lock out.